


Instructions for Neo24 Test

Sample collection

- A maternal blood sample should be collected from the tenth week of pregnancy onwards.
- Collect approximately 10ml of blood sample in the STRECK tubes (camouflage top) provided by Juno Genetics.
- Please use 21-gauge straight needle, not butterfly.
- Seal the STRECK tube well and gently invert at least 5 times.
- Do **not** shake the tube.
- Label the tube with at least two of the following: patient's full name; their unique patient/hospital number; their date of birth.
- Do **not** cover the expiry date of the STRECK tube with the patient details label.

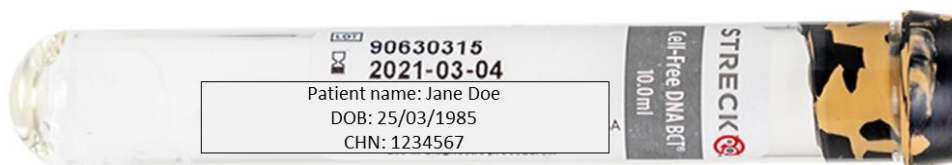

Storage

- Send the samples to Juno Genetics on the same day or, at the latest, the next day after blood collection.
- Before shipment, store the sample either in a refrigerator or at room temperature.
- Do **not** expose the blood sample to temperatures outside the range of 4°C-37°C.
- If storing the sample overnight, refrigeration is recommended, but ensure the sample tube does not come into direct contact with the back or sides of the refrigerator (these may be below 4°C).

Packaging and shipment

- Place the labelled blood tube back in the provided plastic casing.
- Place the completed Neo24 test request form and blood tube in the Neo24 test kit box provided by Juno Genetics.
- For Neo24 test, ship the samples to Juno Genetics at room temperature.

Checklist

- 10ml of maternal blood collected in Juno Genetics STRECK tube
 - tube sealed and labelled as described above
 - tube placed inside the plastic casing in the Neo24 Test kit box
- Neo24 test Request Form completed and placed inside the Neo24 Test kit box
- Consent Form completed